


The "CLENCHER" Custom Fitting Instructions:

DO NOT USE this mouth guard IF you have an under-bite (Class III prognathic jaw relationship).

The "CLENCHER" is a LATEX-FREE product!

FITS EASILY OVER BRACES: After boiling, channel flanges may be widened with thumb. After creating impressions, DO NOT push outside flanges inward against braces/teeth.

OPTIONAL SIZING:

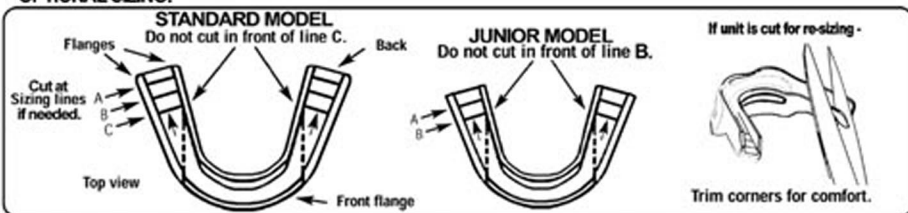

Time critical in seconds.

Time critical in seconds.

2 STAGE BOILING PROCESS

Completely insert Boil-N-Bite Stick as pictured.

FIRST STAGE:
20 SECONDS

Place in water up to boil line.

DO NOT OVER BOIL



SECOND STAGE:
10 SECONDS

Place in water completely.


OVER 30 SECONDS TOTAL DESTROYS THE CLENCHER



AFTER BOILING:

- 1) LEAVING BITE STICK IN BREATHE SPACE, PLACE CLENCHER ON LOWER TEETH FIRST.
- 2) BRING UPPER TEETH INTO TOP CHANNELS.
- 3) BITE FIRMLY FOR 10 seconds.
- 4) RUN UNDER COLD WATER TO SET IMPRESSIONS. REMOVE BITE STICK.

Sanitize regularly with an astringent mouthwash for good hygiene. With proper care, your Brain-Pad should last for a whole season.

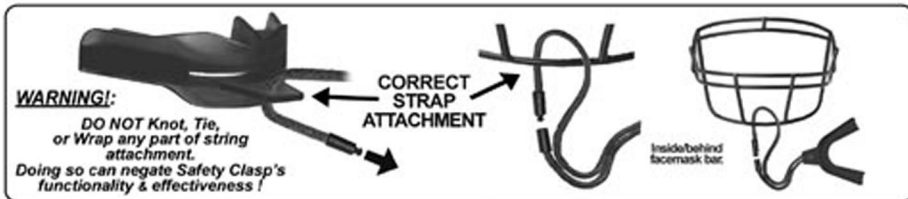

[detach along perforation and KEEP below information to satisfy warranty requirements]

WARRANTY

Brain-Pad's liability for a defective BP is expressly and solely limited to (i) the replacement of the BP; or (ii) the lesser of (a) actual dental expenses incurred by the Purchaser; or (b) the sum of One Hundred Fifty Six (\$156) per injured or replaced tooth, up to maximum liability of Five Thousand Dollars (\$5,000) per incident; provided, however, Brain-Pad's obligations under the immediately preceding sentence shall be reduced by the amount of any insurance collected is payable to Purchaser for the damage, loss, and/or injury claimed by Purchaser on account of the BP. Notwithstanding any other provision in the Warranty to the contrary, (i) this Warranty does not provide or shall be deemed to provide payment or indemnity for orthodontic treatments or preparation of the tooth to be used as an anchor for a bridge, (ii) Brain-Pad shall have no responsibility or liability whatsoever under this Warranty unless (a) the BP was fitted by Purchaser in accordance with the printed instructions accompanying this Warranty; and (b) the original Instructions/Warranty Document received with the purchased BP and a complete written notice from Purchaser of the injury, loss and/or damage is received by Brain-Pad within thirty (30) days after the occurrence of the injury, loss, and/or damage ("Notice") addressed to Brain-Pad Warranty Department, 322 Fayette St., Conshohocken, Pennsylvania, 19428. Other limitations apply. View complete warranty information at: www.brainpads.com.

Manufactured exclusively by Brain-Pad, Inc. Components manufactured in Taiwan. Assembled in the USA. Copyright 2000-2004 Brain-Pad, Inc. All Rights Reserved. Patents: # 5,636,379 # D 493,578. Other patents pending.

Distributed exclusively by: Adams USA. 610 S. Jefferson Ave. Cookeville, TN 38501. 1-800-251-6857. www.adamsusa.com

ADULT WITH STRAP



WCL-205 Blue

INST-11 01 04